

SoliDoser[®] Bullet Points

- **Read the instruction manual!**
- **Keep the 10-dose clips sealed in the foil pouch until ready to use.**
- **Since SoliDose[®] implants are biologics, they should be kept refrigerated. In order to keep moisture from entering the pouch, let the pouch come to ambient temperature before opening and removing clips. After removing a clip, re-seal the Ziploc to again help keep out excess moisture. Remove one clip at a time.**
- **Do NOT store the SoliDoser loaded with product in the refrigerator. The pellets in the clip will absorb moisture and swell. When used again, the device will jam and not function properly.**
- **Never advance the SoliDoser without a clip in the chamber.**
- **If the clip fails to automatically advance, you can use an instrument (pen, probe, etc.) to advance manually until you are finished working cattle.**
- **Never use excessive force to operate the device - you could bend the push-rod or break internal parts.**
- **Do NOT disassemble the SoliDoser. It will void any warranty.**
- **Never apply lubricants to the SoliDoser.**
- **To clean the SoliDoser, use a cloth moistened with water or alcohol as needed. Do NOT immerse the device!**
- **Upon removing the needle and still-extended plunger rod from the injection site, dry-wipe debris before releasing the handle. Disinfect and dry the needle between doses. PREVENT MOISTURE FROM GETTING INSIDE THE NEEDLE.**
- **Upon pulling the SoliDoser trigger do NOT apply resistance to the trigger while it returns to its original position. If resistance is applied, the clip may not advance to the next dose.**
- **Remember: The proper subcutaneous injection sites are in the NECK or the BASE of the ear, NOT the middle-third of the ear.**