



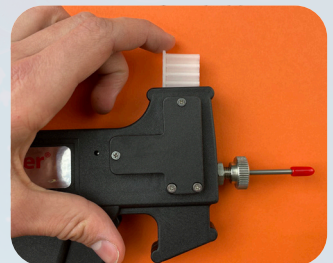

# Use and care of the SoliDoser® and SoliDose® dual-dose pellets

- 01** Insert base of needle into the sleeve, and advance until fully seated and bevel is set correctly.
- 02** Tighten sleeve until the needle is secure.
- 03** Loaded pellet clips should be stored in their foil pouch and refrigerated until ready to use. When pouch is removed from the refrigerator or cooler, lay it flat and allow it to come to outside temperature before opening and removing a clip. After removing a clip, re-seal the pouch to help keep out excess moisture. Remove only one clip at a time.
- 04** Open the package at the tear notch and remove one 10-dose clip. Insert the loaded clip into the top of the SoliDoser and advance until you feel it “click” into place. **DO NOT** pull trigger until the clip is fully advanced!
- 05** Restrain animal and locate injection site in either the neck or in the **BASE** of the back of the ear.
  - A.** If administering in the neck, vaccinate under skin **NOT** in the muscle. “Tenting” the skin before inserting needle will aid in proper administration.
  - B.** If administering behind the ear, insert point of needle under the loose skin in the notch of cartilage on the back side of the ear, **NOT** in the middle third or tip of the ear.

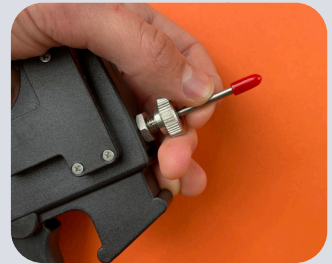

**06** After inserting the full length of the needle, squeeze the trigger to implant the pellets. Keep the trigger depressed until you fully withdraw the needle to make sure the pellets stay in place.



**07** Use a dry, clean cloth to wipe any debris from the tip of the plunger rod BEFORE releasing the trigger, then release trigger to automatically advance to the next dose. Disinfect and dry the OUTSIDE of the needle between doses. Do NOT allow moisture to get inside the needle.



**A.** In the unlikely event that a clip would not advance, it can manually be advanced using a probe or pen as a tool.

**B.** Never use excessive force to operate the SoliDoser.



**08** Do not apply any resistance to the trigger as it is returning to its original position. Doing so may prevent the clip from advancing to the next dose.



**09** The clip will automatically eject after the 10th dose is administered. Do NOT store a loaded clip in the device as pellets may swell and jam the gun. Always protect gun and product from moisture.

**10** Do NOT store the SoliDoser in the refrigerator.

**11** Clean the SoliDoser with a damp cloth, moistened with water or alcohol as needed. Do NOT immerse the SoliDoser and do NOT apply any lubricant.



**12** Do not disassemble the SoliDoser.



SoliDose and SoliDoser are registered trademarks of SolidTech Animal Health, Inc. (Newcastle, OK).



1525 Bioscience Drive Worthington, MN 5618

info@cambridgetechnologies.com

www.cambridgetechnologies.com

Newcastle, OK 73065

405-387-3300 • 800-687-6497

www.solidtechah.com